

MR compatible camera M12

User manual


Articulated arm mount

This manual describes the mounting option articulated arm for the MR compatible camera M12.

Description

With the articulated arm the camera can be mounted very flexible to a variety of objects as different head coils etc.

If not pre-assembled, a plastic adapter plate has to be mounted to the camera with two small M2 brass screws before you can use the camera with the articulated arm.

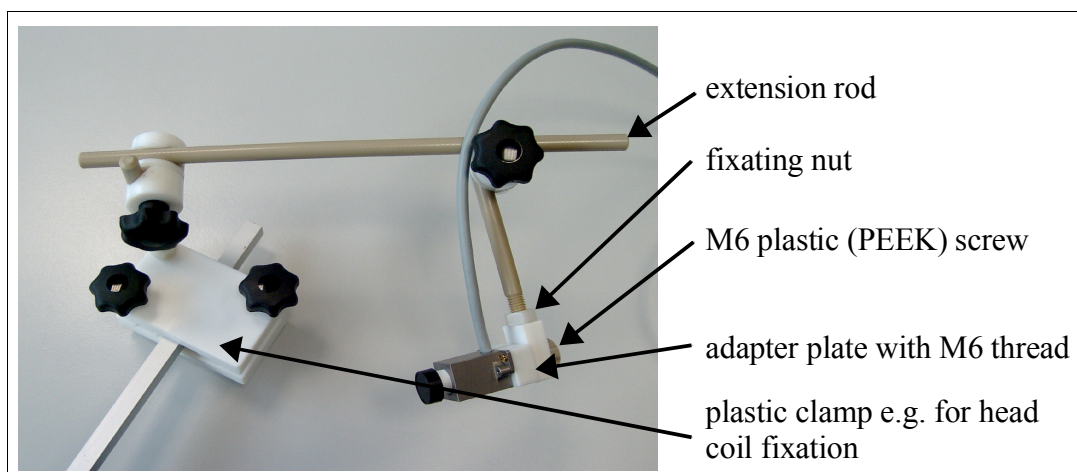

*Figure 1:
Application
sample:
Camera
mounted on
articulated arm*

The white plastic clamp can be mounted to a variety of head coils or similar objects. (see figure 1) With the two articulated joints, the extension rod and the camera rod the camera can be positioned and fixed at the needed position. The plastic plate at the end of the camera rod is mounted to the adapter plate with the M6 plastic (PEEK) screw.

Contact

MRC Systems GmbH
Hans-Bunte-Strasse 10
D-69123 Heidelberg
Germany
phone: +49-6221-13803-00
fax: +49-6221-13803-01
mail: info@mrc-systems.de

MR_CAM_12M- Articulated_Arm_Mount_Manual_v1.sxw	SF - 10/20/06	page 1 of 1
	version: 1	status: approved